# XL-10 ActiLink™ Users Manual

English | French | Spanish

# What Does This Work With?

Use your ActiLink™ USB Transceiver with these

**Wellness  
Connected™** products:

**XL-20/ActiPed™**  
Wireless Activity  
Monitor

**UA-851THW/UA-  
851TMW/UA-851THX**  
Wireless Automatic  
Blood Pressure  
Monitor

**UC-324THW/UC-  
324TMW/UC-324THX**  
Wireless Precision  
Scale

# Getting Started

---

Before plugging the ActiLink™ USB Transceiver into your computer:


1. Go to [www.wellnessconnected.com/getstarted](http://www.wellnessconnected.com/getstarted)
2. Choose the service plan that is best for your needs.
3. Purchase that plan.
4. Download the Wellness Connected software that is appropriate for your computer.
5. Run the software and follow the installation process.
6. Enter in the required fields (your health information and product serial numbers).
7. Choose your widgets.
8. Set your goals.
9. Start using Wellness Connected!

# Finding The Serial Number

Top

Bottom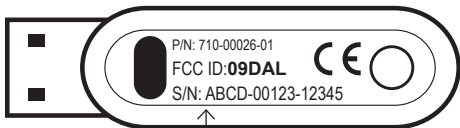

This is the number you need to enter  
at [www.wellnessconnected.com](http://www.wellnessconnected.com)

# Frequently Asked Questions

**Q: Why can't I store data files or photos on this memory stick?**

A: The ActiLink™ USB Transceiver is not a memory stick. It is a wireless transceiver (antennae) for receiving signals from Wellness Connected devices.

**Q: I purchased an XL-20 Wireless Activity Monitor and now I have two ActiLinks, what do I do with the second one?**

A: Use it in another computer to upload your data in multiple locations, such as your home or office.

**Q: I put the ActiLink into my computer's USB port, why aren't I getting data from my Wellness Connected devices?**

A: Have you fully installed the software? Have you purchased a service plan? Have you completed installation and set-up? Is the distance between your devices and the ActiLink too large or obstructed by metal? If none of these are the issue please contact us and we'll help.

# Warranty

This ActiLink™ USB Transceiver is covered by a 1 year warranty.

A&D Medical (the "Company") warrants to the first retail purchaser that this A&D Medical ActiLink USB Transceiver (the "Product") will be free from defects in material and workmanship for one year from the time of purchase. To obtain warranty service for such defects, return the Product, freight and insurance prepaid, together with satisfactory proof of date of purchase such as a sales receipt and shipping costs to:

**A&D Medical**  
**Attn: Warranty Department**  
**1756 Automation Parkway**  
**San Jose, CA 95131**

Include a check for return shipping and insurance. Contact A&D Medical at 1-888-726-9966 for appropriate shipping costs.

At A&D Medical's option, the Company will repair or replace any Product found to be defective and return it to you. (Any replacement product may be new or refurbished.) This warranty will not apply if the defect or malfunction results from: subjecting the Product to any but the specified voltage; modification, alteration or repairs of the Product by persons not authorized by A&D Medical; misuse, abuse, damage during transit or lack of reasonable care with respect to the Product; or failure to follow the written instructions enclosed with the Product.

THE WARRANTY AND REMEDY PROVIDED ABOVE ARE EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IN NO EVENT SHALL A&D MEDICAL BE LIABLE FOR ANY LOSS, INCIDENTAL, OR CONSEQUENTIAL DAMAGES, RESULTING FROM THE USE OR INABILITY TO USE THE PRODUCT, WHETHER RESULTING FROM BREACH OF WARRANTY OR ANY OTHER LEGAL THEORY, EVEN IF A&D MEDICAL HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH INCONVENIENCE, DAMAGE OR LOSS.

Some states do not allow the exclusion of incidental or consequential damage, so that the above exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that may vary from state to state.

A&D Medical assumes no responsibility with regard to the use of this product. Every effort has been made to ensure that the information in this manual is accurate. A&D Medical is not responsible for printing or clerical errors.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference and (2) this device must accept any interference received, including interference that may cause undesired operations. Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this device.

# Contact Information

---

For more information regarding use, care or servicing of your XL-10 ActiLink™ USB Transceiver, contact:

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# Important!

---

If you need assistance with the set-up or operation, we can help! Please call us **FIRST** before contacting your retailer at A&D Medical Customer Service Line 1-888-726-9966 (Toll Free). A specially trained representative will assist you.

# **A&D**

**A&D Medical**

**A division of A&D Engineering, Inc.**

**1756 Automation Parkway**

**San Jose, CA 95131**

**[www.andmedical.com](http://www.andmedical.com)**

© 2010 A&D Medical. All rights reserved.

LifeSource, the LifeSource logo, Wellness Connected, and the Wellness Connected logo are trademarks of A&D Medical registered in the U.S.

THE XL-10 ACTILINK USB TRANSCEIVER is covered by one or more of U.S. patent numbers 6336365, 6018705, 6052654, 6493652, 6298314, 6560903, 6611789, 6536139, 6876947, 6882955, 7187924 and patents pending.

SPECIFICATIONS SUBJECT TO CHANGE WITHOUT NOTICE.

Printed in China 11/2010

I-MAN-XL10 Rev. A

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**Una división de A&D Engineering, Inc.**